

COMPACT TOILET RAIL

LVA1055



vive
vivehealth.com



To see all FAQs
in one place,
visit vhealth.link/1beba

OWNER'S MANUAL

COMPACT TOILET RAIL

WHAT'S IN THE PACKAGE



4x Locking Screws



2x Rear Bracket
Screw Knobs



Rear Bracket



2x Padded Armrests
(Left & Right)



2x Slide Assist Rails
(Left & Right)



2x Toilet Seat Rester
(left & Right)

ASSEMBLY AND INSTALLATION INSTRUCTIONS




Want to watch us do this?
Check out vhealth.link/d134e for an
easy instructional video

NOTE: The Rear Bracket, Arm Rests, Assist Rails, and the Seat Resters are all labelled to help identify which pieces fit together. Fit matching numbers and matching letters together for proper fit.

1. Slide the end of the Toilet Seat Rester labelled “1” over the end of the Armrest with the matching “1” label.



Fasten the two pieces together using a Locking Screw, insert the screw through and make sure the hexagonal head matches the hexagonal cutout of the opening.

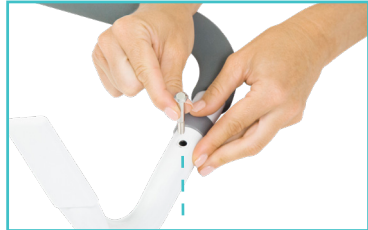

Add the brass pin, narrow end first, to the opposite end and make sure it fits into the screw hole.



Then add the plastic washer and finally screw on the nob, tighten securely.

Repeat Step 1 with the Toilet Seat Rester and Armrest labelled “2”.



NOTE: Make sure you place the bolt so that the hexagon shaped head fits and LOCKS into the hexagon shaped hole. Your brass piece will then lock in on the other end in the normal shaped round hole. Please double check the picture to make sure you have this correct.

2. Insert the end of the Armrest labelled “A” into the end of the Slide Assist Rail with the matching “A” label.



Fasten the two pieces together using a Locking Screw, insert the screw through and make sure the hexagonal head matches the hexagonal cutout of the opening.



Add the brass pin, narrow end first, to the opposite end and make sure it fits into the screw hole.




Then add the plastic washer and finally screw on the nob, tighten securely.

Repeat Step 2 with the Armrest and Slide Assist Rail labelled “B”.

3. Insert the unfastened end of the Slide Assist Rail labelled “A”, into the end of the Rear Bracket with the matching “A” label.

Make sure that the wide, flat part of the Rear Bracket is pointed towards the Toilet Seat Resters.


Align the holes and tightly secure at the desired width using a Rear Bracket Screw Knob.

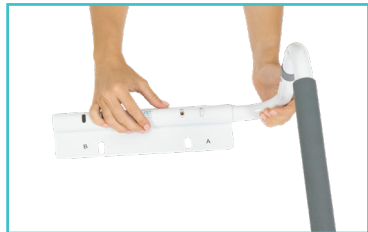

Repeat Step 3 with Slide Assist Rail and Rear Bracket side labelled “B”.



4. Remove the seat and lid from the toilet and set it aside along with all of the attaching hardware.

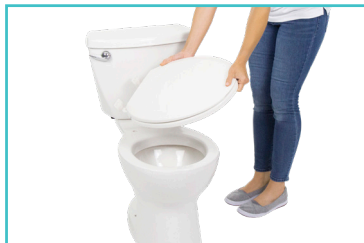

5. Place the Safety Rail on top of the toilet bowl so that the holes in the wide, flat section of the Rear Bracket are lined up with the seat/lid holes in the toilet seat.



6. Make sure that the flat side of the Rear Bracket is fitted between the toilet bowl and the seat. (TOILET SEAT HARDWARE NOT INCLUDED)



Reattach the toilet seat and lid to the toilet through the Rear Bracket holes and hand tighten the entire assembly down.



7. Make sure that the grip pads on the Toilet Seat resters are making good contact with the rim of the toilet. If necessary, adjust the width of the Safety Rail using the Rear Bracket Screw Knobs.



CAUTION: The Compact Toilet Rail is meant to be used as a safety frame and stability assist only, it is NOT designed or intended to bear an individual's full body weighing. Periodically check all of the screws and fasteners to make sure that the rail is securely attached, re-tighten if necessary. Failure to use the Safety Rail as intended may result in damage or personal injury.

MAINTENANCE

The only maintenance required for your Compact Toilet Rail is some light cleaning with a damp cloth and common bathroom cleaners. After cleaning, make sure that the Seat Resters and the porcelain beneath are completely dry before using the rail as moisture can prevent the tips from gripping the toilet. As dirt and moisture can collect beneath the Rear Bracket, removing the Safety rail periodically for a more thorough cleaning is recommended.

If you have any problems with your Safety Rail, including installation, use, or maintenance, please contact service@vivehealth.com.

WARNINGS

300lbs weight limit!

When installed properly the arms in the device will not move at all. If your arms are able to swing out of place from above the toilet seat check the manual again and re-trace your steps because you did something incorrectly in the installation.

Do not use this device if you can freely move the arms or if the device has not been secured to the back of the toilet seat as instructed.

HAVE MORE QUESTIONS?

Check out our list of Frequently Asked Questions at
[vivehealth.link/1beba](https://vivehealth.com/health.link/1beba) for helpful answers.



And if that doesn't answer your question, our
customer service team would love to help!
Feel free to connect with them by phone,
e-mail, or chat on our website

service@vivehealth.com
1-800-487-3808
vivehealth.com

Distributed by

vive
health

8955 Fontana Del Sol Way
Naples, FL 34109
Made in China